

Installing Drivers for 300-2/3A

1. Uninstall the existing version of CardioScan Premier by going to Start -> Settings -> Control Panel -> Add or Remove Programs -> select Holter Software -> Remove.
2. Download and run the 300-2/3A drivers installer located on our website at www.holterdms.com for your desired version.
3. Download and run the CardioScan Premier patch located on our website for your installed version.
4. Run CardioScan Premier from the shortcut on your desktop.

Enrolling Patients on the 300-2/3A

1. Make sure there is no battery in the recorder.
2. Connect the smaller end of the download cable into the connector located behind the rubber tab on the recorder.
3. Connect the larger end of the download cable into a USB port on the computer.
4. Run CardioScan Premier, and click on Enroll.
5. Type in the patient's information, and click Next.
6. After the message "DMS 300 is formatted" appears, click OK. Click OK again.
7. Remove the download cable.
8. Connect the patient's lead wires to the recorder.
9. Insert a battery, and the recorder will begin recording.

Downloading Patients from the 300-2/3A

1. Remove the battery from the recorder.
2. Disconnect the patient's lead wires from the recorder.
3. Connect the smaller end of the download cable into the connector located behind the rubber tab on the recorder.
4. Connect the larger end of the download cable into a USB port on the computer.
5. Run CardioScan Premier, and click on New Patient.
6. Type in the patient's information, and click Next.
7. When the download is complete, the "Set ST and QT" window will appear.