


CardioScan Premier Client User Manual V1.6

CardioScan Premier Client is a special version which allows the client at a remote location to extract a patient's data from a Holter device and upload it to the Satellite Server for the technicians to download and analyze.

1. Installing CardioScan Premier Client

Note: If you have already installed the Satellite Client, then CardioScan Premier Client is already installed. The two programs are installed together.

- 1.1 To begin the installation of the CardioScan Premier Client software on the Client PC, locate the file "SatelliteClientAndHolterDownloadSoftware.exe" and double-click it (if you already have CardioScan Premier 12 installed, you can skip this step).

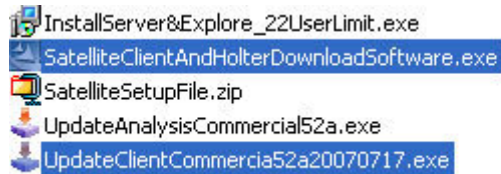

- 1.2 After installation is complete, plug in the USB security key labeled "SAT". Wait a few seconds for the computer to recognize it, and the green light on the key should remain on.
- 1.3 Next, find "UpdateClientCommercial5xxxxxxxxxx.exe" and double-click it (the x's can represent any number or letter depending on the current version). This will patch your Satellite Client software to the latest version. Repeat this step for future updates.

- 1.4** The Satellite Client software will be installed as part of the CardioScan Premier Client version. Follow the on-screen instructions until installation is complete.




- 1.5** After the installation is complete, run the CardioScan Premier software like you normally would. To verify that the client PC is running the correct version, it should say "Client Download" above the Version number on the top right corner.




2. Using CardioScan Premier Client



- 2.1** After creating a new patient and importing the data from the Holter device or flash card, the Satellite Client running in the background will automatically upload it to the Satellite Server for analysis.
- 2.2** When analysis is complete, the technician will send the completed report back to the Satellite Server, and the Satellite Client will automatically download the completed report. Click "Refresh" in the bottom-right to check for newly downloaded files.

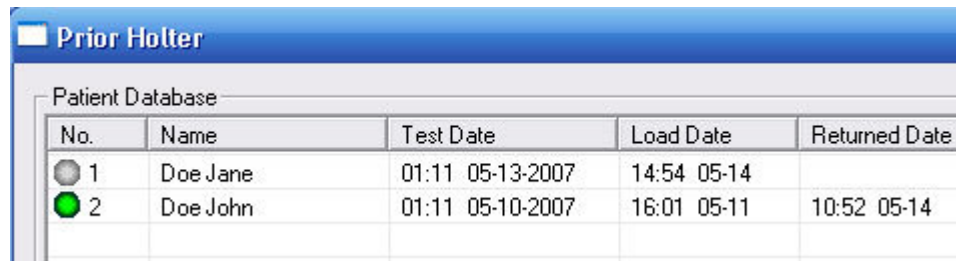You can check the status of various patients by the colored ball next to their name:

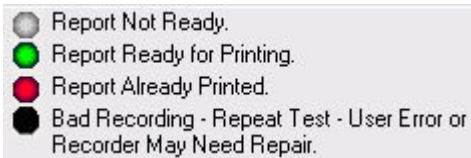

No.	Name	Test Date	Load Date	Returned Date
1	Doe Jane	01:11 05-13-2007	14:54 05-14	
2	Doe John	01:11 05-10-2007	16:01 05-11	10:52 05-14



	Report Not Ready.
	Report Ready for Printing.
	Report Already Printed.
	Bad Recording - Repeat Test - User Error or Recorder May Need Repair.

Grey Ball: Patient data has been uploaded to the Satellite Server. No access is allowed yet.

Green Ball: Patient data is returned from the Satellite Server and is ready for access.

Red Ball: Patient data has been printed from the current PC.